


How to use Tigran™ PeriBrush™




Remove excessive granulation tissue before using the brush.

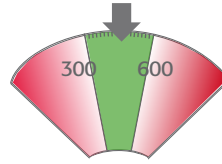


Use the brush at preferred speed of 300–600 rpm.



Continuous irrigation with sterile saline is recommended. Keep brush in close contact with implant surface.

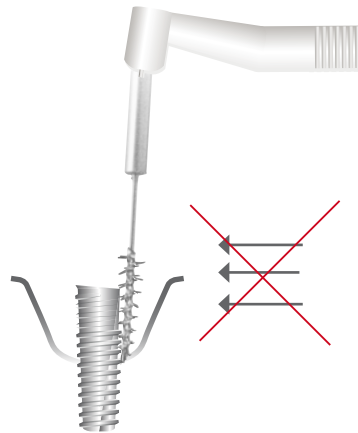

Use gentle movements without excessive force.

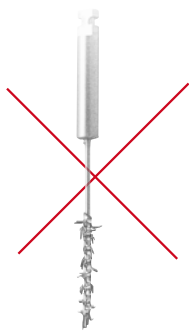

Do not use if damaged.



Do not reuse. Do not resterilize. Risk for organic contamination.

TIGRAN TECHNOLOGIES AB

- MEDEON SCIENCE PARK, 205 12 MALMÖ, SWEDEN
- WWW.TIGRAN.SE • INFO@TIGRAN.SE • +46 40 693 92 70

